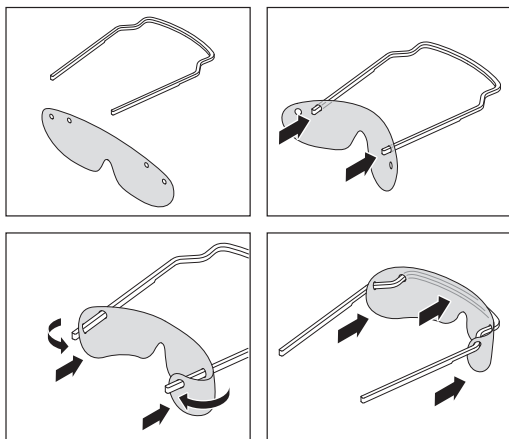
DURABLE

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EN Operating Instructions

Replacement foils for the DURABLE eye protective visor 3435. The shields for the eye protective equipment can be exchanged. This article complies with Regulation (EU) 2016/425 Category II on personal protective equipment (PPE), is type-tested and CE marked. For safe use of this article, the user must have read and understood the instructions for use before first use.

Use

This article protects against splashes of liquids coming towards the eyes. This eye protection device does not provide protection against mechanical impact. Dispose of after use. Reuse of the temple is at the discretion of the user. We recommend using the temple for a maximum period of 2 years. The duration of use depends on various factors such as use, cleaning, storage and maintenance. The films are only suitable for single use.

Storage

Storage: in original packaging, in a dry environment.

Storage conditions: approx. 18°-23° C, approx. 55% humidity

Period of use: Do not use the article for 2 years after the date of purchase.

Cleaning and disinfection of the spectacle frame:

With clear, lukewarm water and household detergent (no alkalis, acids or solvents). The article can be disinfected with commercially available disinfectants or sterilised in a steam steriliser. Avoid strong mechanical cleaning.

Risks

This article does not provide protection against mechanical impact. It does not provide protection against types of radiation (e.g. solar radiation, UV, welding, IR). The eye protection device is only for use at room temperature and is not fire resistant.

Do not use a damaged eye protector, it may affect vision and protection. If the eye protection device is damaged, it must be replaced (sharp edges can cause injury). Materials that come into contact with the wearer's skin could cause allergic reactions in sensitive individuals.

Declaration of conformity

The declaration of conformity can be found at: durable.eu/3435CE

Certification authority:

BSI Group The Netherlands B.V.

Say Building, John M. Keynesplein 9, 1066 EP

Amsterdam, Netherlands

Notified Body 2797

The article complies with the following Union regulations: Regulation (EU) 2016/425 Personal Protective Equipment (PPE) and Directive 2001/95/EC General Product Safety.

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